## DEUTSCH

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Ceramir® Crown & Bridge ist eine permanente, röntgensichtbare Keramikzement. Der main component in the powder is calcium aluminate and in the liquid, water, the cement, which hardens as a result of a chemical reaction, is self-setting. In Ceramir Crown & Bridge QuikMix, the powder is supplied in a jar and the liquid is supplied in a drop bottle. The minimum net mass of powder is 15g and the minimum net volume of liquid is 10 g (9 ml).

**ACCESSORIES:**  
• A scoop for powder dosing is included with powder  
• A non-absorbing mixing pad  
• A plastic spatula for cement protection

**2. INDICATIONS FOR USE**

Ceramir Crown & Bridge ist intendiert für permanent cementation of:  
• Metal and porcelain fused to metal crowns and bridges  
• Gold alloys or porcelain  
• High-fused ceramic crowns and bridges suitable for conventional cementation (e.g. zirconia, alumina, and lithium disilicate)

**Contraindications**

• Do not use as a direct pulp-capping agent, in case of close proximity to the pulp, use accepted standard clinical procedures.

**Precautions**

Ceramir® Crown & Bridge contains polyacrylic acid, which may be irritating to eyes and skin. Avoid contact with eyes. In case of skin contact, rinse with plenty of water. In case of contact with eyes, rinse immediately with water and seek medical advice.

• Do not use on patients with known allergy to polyacrylic acid. If such reactions are observed, discontinue the use of the product and consult a physician.

**3. STEP-BY-STEP INSTRUCTIONS**

**ATTENTION** DO NOT USE ANY PRE-TREATMENT AGENTS, E.G. SILANES, PRIMERS OR BONDING AGENTS WHEN CEMENTING WITH CERAMIR CROWN & BRIDGE. THE USE OF ANY SUCH PRE-TREATMENTS MAY COMPROMISE THE BONDING TO THE CEMENT.

**3.1 Tooth preparation**

• Remove the temporary restoration and cement.  
• If needed, clean the tooth using pumice or prophylaxis paste (with or without fluoride).  
• Rinse with water.  
• Air dry gently with oil free air or cotton pellets. Do not desiccate.

• Isolate the site with cotton rolls.

**3.2 Preparation of the restoration**

**Lithium Disilicate**

Etch with HF acid according to manufacturer's instructions. Do not use any pre-treatment agents, e.g. silanes, primers or bonding agents.

**All other restorations**

Do not use any pre-treatment agents, e.g. silanes, primers or bonding agents when cementing with Ceramir Crown & Bridge. Use of any such pre-treatments will destroy the bond between cement and restoration.

**3.3 Pulp protection**

• If pulp protection is required, use accepted standard clinical procedures.

**3.4 Powder-liquid dispensing and mixing**

The standard powder to liquid ratio is 1,8/1,0 (w/w) (1 level scoop of powder to 2 drops of liquid). Mixing pad type: Non-absorbing mixing pad or glass slab.

• Dispense the desired number of level powder scoops, using the scoop provided, onto a mixing pad/slab.

• Hold the liquid bottle vertically and squeeze gently to dispense liquid (2 drops per powder scoop). For more consistent results, remove the nozzle there is no air in the nozzle while dispensing e.g. by discarding the first few drops before holding the bottle upside down.

• Close bottles immediately after use.

• Add all the powder to the liquid using a plastic spatula.

• Mix to a homogeneous paste (maximum 30 seconds).

**Working and setting times**

• Working time (from start of mixing) at 23°C/73°F: 2 minutes.

• Setting time (from end of mixing): 3 to 8 minutes.

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**Limitation of Liability**

If this Product is used ineffective within the Warranty Period, your exclusive remedy is Doxa Dental AB's sole obligation and liability shall be the repair or replacement of the Doxa Dental AB Product, EXCEPT WHERE PROHIBITED BY LAW. DOXA DENTAL AB WILL NOT BE LIABLE FOR ANY LOSS OR DAMAGE ARISING FROM THE PRODUCT, WHETHER DIRECT, INDIRECT, SPECIAL, EXEMPLARY, PUNITIVE OR CONSEQUENTIAL, INCLUDING LOSS OF PROFITS, REGARDLESS OF THE THEORY ASSERTED, INCLUDING WARRANTY, BREACH OF CONTRACT, NEGLIGENCE OR STRICT LIABILITY.

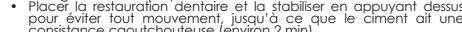
**Trademarks**

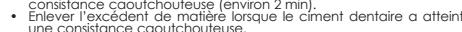
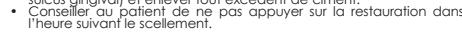
Ceramir® is a trademark of Doxa Dental AB.

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Ceramir® er et varemærke for Doxa Dental AB.