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**FOR PROFESSIONAL DENTAL USE OR RECOMMENDED INDICATIONS ONLY (SEE 2. INDICATIONS FOR USE).**

### 1. Product description

Ceramir® Bico ceramic Implant Cement is a permanent, radiopaque bioceramic cement. The main component in the powder is calcium phosphate (calcium phosphate). The cement, which hardens as a result of a chemical reaction, is self-setting.

In Ceramir Bico ceramic Implant Cement QuikCap, the cement is supplied in a ready-to-use pre-filled cartridge. The minimum volume of mixed cement delivered from one capsule is 0,17 ml.

For use with the Ceramir Bico ceramic Implant Cement, you will need a Ceramir Bico ceramic Applicator and Ceramir Applicator 2 from Daxa Dental AB.

### 2. Indications for use

Ceramir Bico ceramic Implant Cement is intended for permanent cementation of dental implants. The cement, which hardens as a result of a chemical reaction, is self-setting.

Ceramir Bico ceramic Implant Cement is intended for permanent cementation of dental implants. The cement, which hardens as a result of a chemical reaction, is self-setting.

- Metall and porcelain fused to metal restorations
- High strength ceramic restorations suitable for conventional cementation (e.g. alumina, alumina and lithium disilicate)

### Precautions

Ceramir Bico ceramic Implant Cement contains polycrylic acid, which can be irritating to eyes and skin. Avoid contact with eyes. In case of contact, flush thoroughly with water. Rinse immediately with eyes, rinse immediately with plenty of water and seek medical care.

- Do not use on patients with known allergy to polycrylic acid. In rare cases, allergic reactions have occurred in some persons. If such reactions are observed, discontinue the use of the product and consult a physician.

### 3. Step-by-step instructions

**Note:** Do not use any pre-treatment agents, e.g. slanes, primers or bonding agents when cementing with Ceramir Bico ceramic Implant Cement. Use of any such pre-treatment agents may compromise the bonding of the cement.

- Temporary preparation
  - Remove implantation unit and rinse with water.
  - Air dry for 30 seconds.
  - Dry the operation field with for example cotton rolls.
- Preparation of the restoration

### Lithium Disilicate

Eth with HF acid according to manufacturer's instructions. Do not use any pre-treatment agents, e.g. slanes, primers or bonding agents.

All other restorations

Do not use any pre-treatment agents, e.g. slanes, primers or bonding agents when cementing with Ceramir Bico ceramic Implant Cement. Use of any such pre-treatment agents may compromise the bonding of the cement and the restoration.

At all steps, care can be used if recommended by the restoration manufacturer.

**ATTENTION: WORK QUICKLY** Start mixing immediately following activation (the reaction between the liquid and powder starts directly after activation).

- Apply cement
  - Cover the restoration with the applicator. Push the minimum volume of mixed cement delivered from one capsule into a hard, flat surface of the plunger until it stops (Fig. 1).
  - Remove the applicator. Bring the applicator into body of the capsule using holding cap (Fig. 1).

### 3.4 Mixing

- Mix the powder and capsule in a high-speed oscillating (4000 to 5000 rpm) or rotating capsule mixer (Fig. 2).
- Mix for 30 seconds.

### 3.5 Application

- Apply the cement to the capsule into the applicator using the grooves to hold it in place (Fig. 4).
- Genly cise the doing level (Fig. 3a) two times to push the cement level into the capsule.
- Then apply the cement by gently squeezing the lever two more times. (Fig. 5)
- After application, remove the empty capsule using the applicator release button (Fig. 3b)

**Working and setting times**

- Working time (from end of mixing) at 23°C/73°F: 2 minutes.
- Setting time (to reach the maximum strength) is 3 to 8 minutes.

**Note:** Room temperatures above 23°/73°F will accelerate setting and reduce the working time available.

### 3.6 Seating and finishing

Remove the excess cement under pressure. To avoid any movements, until the cement reaches a rubber like consistency (approx. 2 min) (Fig. 4-7).

Remove excess material when the cement has reached the rubber-like consistency (Fig. 8).

Keep the restoration stabilized for an additional 4 minutes (Fig. 9). This will ensure the strength of the reaction (especially the grafted silica) and remove any remaining cement.

Advise the patient not to load their restoration for the first hour after cementation.

### 4. Cleaning the applicator

Always clean with standard cleaning agents, e.g. alcohol-based spray disinfectants.

Clean Ceramir Applicator 2 with alcohol soaked gauzes. Do not use chemical sterilants. Disinfectants can cause damage to the capsule.

AUTOCLAVE: Autoclave up to a maximum temperature of 140°C. Follow autoclave manufacturer's instructions for use. Do not ultrasonic clean.

### 5. Storage and shelf life

Ceramir Bico ceramic Implant Cement QuikCap capsules of temperatures between 4°C/39°F and 20°C/68°F. Do not open the aluminum pouch until immediately before use.

Do not use Ceramir Bico ceramic Implant Cement QuikCap capsules after their expiry date.

### Warranty

Daxa Dental AB warrants that the Product will be free from defects in material and manufacture. This warranty shall (i) only apply within the Product's shelf life/expiry date (the "warranty period") and (ii) further only if the Product has been used strictly in accordance with the instructions for use. DAXA DENTAL AB MAKES NO OTHER WARRANTIES, EXPRESS OR IMPLIED, REGARDING THE PRODUCT. THE PROVISIONS OF THIS WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR NON-INFRINGEMENT OF INTELLECTUAL PROPERTY RIGHTS. Daxa is not responsible for any third party's misstatements or the accuracy of the Product's information.

**Limitation of Liability** If this Product is found defective within the Warranty Period, your exclusive remedy shall be a replacement of the Product. EXCEPT WHERE PROHIBITED BY LAW, DAXA DENTAL AB WILL NOT BE LIABLE FOR ANY LOSS OR DAMAGE ARISING FROM THE PRODUCT, WHETHER DIRECT, INDIRECT, SPECIAL, PUNITIVE, INCIDENTAL OR CONSEQUENTIAL INCLUDING LOSS OF PROFITS, REGARDLESS OF THE THEORY ASSERTED, INCLUDING WARRANTY, BREACH OF CONTRACT, NEGLIGENCE OR STRICT LIABILITY.

**Trademarks** Ceramir is a trademark of Daxa Dental AB. Date of latest revision: 2019-01-16.

**LOI** Both code      Radiopaque      Capsule      Applicator

Temperature limitation      Capsule      Applicator

Use by      Capsule      Applicator

Consulting number      Capsule      Applicator

Do not use any pre-treatment agents, e.g. slanes, primers or bonding agents when cementing with Ceramir Bico ceramic Implant Cement. Use of any such pre-treatment agents may compromise the bonding of the cement.

The extended symbol glossary is provided electronically at <http://www.ceramidental.com/symbloc/>.

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