# ceramir®

## PROTECT LC SYRINGE

- en** Light-cured resin-modified bioceramic pulp protectant
- es** Protector pulpar biocerámico modificado con resina fotopolimerizable

## Doxa

### Distributed by / Contact

If you have any questions, please contact:

Doxa Dental AB  
Axel Johanssonsgata 4-6  
SE-754 50 Uppsala  
SWEDEN

Phone: +46 (0) 18 478 20 00  
[www.ceramirdental.com](http://www.ceramirdental.com)

**For US customers:**  
Doxa Dental Inc.  
1(855)Doxa – USA (369-2872)  
[www.ceramirdental.com](http://www.ceramirdental.com)

### Symbols

-  Batch code
-  Temperature limit
-  Use-by date
-  Catalogue number
-  Consult instructions for use
-  Caution
-  Do not re-use

Rx ONLY Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.

Rx ONLY Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.

**en**  FOR PROFESSIONAL DENTAL USE IN RECOMMENDED INDICATIONS ONLY (SEE "2 INDICATIONS FOR USE").

## 1. Product description

Ceramir® Protect LC is a light-cured resin-modified bioceramic pulp protectant, designed to act as a barrier and protect the pulp. Ceramir Protect LC can be applied very precisely, even in deep cavity preparations, thanks to its thixotropic behavior combined with the supplied needle tips. A controlled setting is achieved by light-curing of Ceramir Protect LC.

## 2. Indications for use

### 2.1 Direct pulp capping for any pulpal exposures, including:

- carious pulp exposure
- mechanical pulp exposure
- pulp exposure due to trauma

### 2.2 Indirect pulp capping in deep preparations:

- under amalgam restorations
- under Class I and Class II composite restorations
- under cements
- as an alternative to calcium hydroxide

## 3. Precautions

- Because of its high pH, Ceramir Protect LC must not be used in the direct intraoral environment. Do not place on enamel, cavity margins or leave exposed to oral environment.
- The success of the pulp capping procedures may be influenced by preparation contamination. Use of a rubber dam or adequate isolation is recommended.
- Pulpal hemorrhage must be controlled prior to the application of Ceramir Protect LC. If hemorrhage cannot be controlled, consideration should be given for the initiation of endodontic therapy over direct pulp capping procedures.
- Ceramir Protect LC is a light-cured material. Proceed immediately once material has been dispensed.

## 4. Step-by-step instructions

Screw off the cap of the syringe and substitute it by a needle tip. **Store the cap!**

After use of the syringe replace disposable needle tips by the cap. **Needle tips are single use only.**

### 4.1 Direct pulp capping for any pulpal exposures

#### 4.1.1 Cavity Preparation

1. When opening the pulp, first stop any bleeding that may occur. Remove excess moisture with a sterile cotton pellet.
2. Leave the surface mildly moist.

#### 4.1.2 Application

1. Apply Ceramir Protect LC punctiform and small area (with a layer thickness  $\leq 0.5$  mm) to the opened pulp. Cover all exposed areas of the opened pulp with Ceramir Protect LC and extend optional up to 1 mm into dentine.
2. Cure for 20 seconds, using a polymerization unit (wavelength range 400-500 nm) with a light intensity of at least 1000mW/cm<sup>2</sup>.
3. Proceed with the restoration according to the manufacturer's instructions.

**NOTE!** If using a total-etch adhesive, etch only the exposed cavity areas, and not the applied Ceramir Protect LC layer

### 4.2 Indirect pulp capping agent in deep preparations

#### 4.2.1 Cavity Preparation

1. Remove all infected carious tooth structures.
2. Remove excess moisture with a sterile cotton pellet. Leave the surface mildly moist.

#### 4.2.2 Application

1. Apply Ceramir Protect LC small area (with a layer thickness  $\leq 0.5$  mm) directly onto the area closest to the pulp.
2. Cure for 20 seconds, using a polymerization unit (wavelength range 400-500 nm) with a light intensity of at least 1000mW/cm<sup>2</sup>.
3. Proceed with the restoration according to the manufacturer's instructions.

**NOTE!** If using a total-etch adhesive, etch only the exposed cavity areas, and not the applied Ceramir Protect LC layer

## 5. Storage

Store at 4 - 25 °C (39 - 77 °F) protected from light and moisture. Close syringe, using the cap, immediately after use. Do not use after expiry date.

## 6. Additional Notes:

- Ceramir Protect LC contains polymerizable methacrylate monomers. Avoid prolonged or repeated exposure with skin, oral soft tissues, and eyes, since this may result in irritation or possible corneal damage.
- In case of contact with eyes, rinse with copious amounts of water and consult a physician immediately. In case of skin contact wash immediately with water.
- Susceptible individuals may experience skin rash, oral mucosa irritation, or other allergic reactions (allergic contact dermatitis).
- Ceramir Protect LC should not be taken orally.
- Ceramir Protect LC is not recommended for use with patients who have a history of allergic reactions to methacrylate resins.
- Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the material. Neither store the material in proximity of eugenol containing products, nor let the material allow coming into contact with materials containing eugenol.
- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.

Ceramir® is a trademark of Doxa Dental AB.  
Date of latest revision: 2021-03-09  
[www.ceramirdental.com](http://www.ceramirdental.com)

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Ceramir® es una marca comercial de Doxa Dental AB.